


Instructions for Use

BendEase Hand with Flo-Form™ - Universal

Indications for use: Provides moderate, comfortable support of the hand and wrist, post injury or surgery.

Product Features

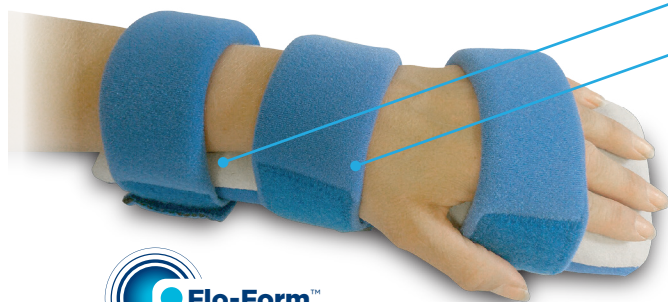

Flo-Form™ Fluidized Medium: Provides even pressure distribution at points of orthotic contact.

Straps: Sized to provide a clear view of the hand, and allow easy access for procedures like IVs and pulse oximetry.

Thermal Adaptive Technology: For optimal comfort.

Foam Over Aluminum Base: Can be remolded by your health professional to continue restorative process as the patient improves.



Instructions for Use:

Please read all of the instructions before use. Use as directed by clinician. Always follow your facility protocol for skin care.



1. Remove the hand splint from the bag and open the straps.



2. Position the hand in the splint.



3. Wrap the straps around hand/wrist/forearm and hook other end of strap to blue fabric. Shown with two options for the finger strap to either go over the ends of the fingers, or over the back of the hand.



4. Run your finger under each strap to make sure it is not too tight. If the straps are too tight, signs of discoloration or swelling may be present.

Instructions for Use

BendEase Hand with Flo-Form™ - Universal

Cleaning Instructions:

Use hospital-approved disinfectant or household antimicrobial cleaner/disinfectant according to the label instructions for the rigid splint base. Wipe the fluidized insert with the same chemical. Padding can be laundered by hand or on gentle cycle of home type washer with mild soap. Rinse. Squeeze between towels to remove excess water. Line dry. Do not use chlorinated products such as bleach. Afterwards, reassemble to original places, making sure the fluidized insert is evenly distributed.

Warranty:

See restorativemedical.com for warranty information.

Warnings:



- This device is for single patient use only.
- Consult a clinical professional before using this splint.
- Check all skin areas each time the splint is removed. If any discoloration does not return to normal within 3-5 minutes, discontinue use and report to your health professional. Discontinue use of the splint until this situation has been resolved.
- Do not puncture the Flo-Form™ fluidized insert in straps or use sharp objects nearby.

Recommended Wear Time:

Apply the splint initially for 30 minutes. Remove and check skin for redness or breakdown. If skin integrity is compromised, discontinue use and contact your health professional. Otherwise, gradually increase wearing times to the duration recommended by your clinician to no more than 6 hours at a time. Restorative Medical's wear times are suggested and should be discussed with your health professional.

Cautions:

- Not user repairable. Consult your clinician or Restorative Medical if damaged.
- No hazardous material present. Follow local environmental codes for waste disposal.
- If punctured, the Flo-Form™ fluidized inserts should be placed in sealed plastic bags to avoid spillage of the fluidized material.
- Do not autoclave or microwave.
- Replace the Flo-Form™ insert promptly if punctured or damaged.



This product is not made with natural rubber latex (NRL) and di-(2-ethylhexyl) phthalate (DEHP).



Prevent • Protect • Restore

332 East Broadway • Brandenburg, KY 40108
(800) 793-5544 • 270-422-5454 • Fax: (270) 422-5453
www.restorativemedical.com



RM- BendEase Univ Fluid IFU v1.0 - 11/1/2017