

## Instructions for Use

### Restorative Hyperextension Stop Cervical Attachment to TLSO

**Indications for use:** Provides support and helps to manage head alignment and upper-body posture.

#### Product Features



- Padded hyperextension roll holds the head comfortably.
- Wide platform securely holds the hyperextension roll.
- Flex Technology vertical slat is flexible to gradually bring the neck to the vertical position.
- High-temperature plastic can be remolded by your health professional to position the patient's head forward gradually.

#### Instructions for Use:

Please read all of the instructions before use. Use as directed by your clinician. Always follow your facility protocol for skin care.

1. After fitting the TLSO and assisting the patient to a sitting position with the buttocks as far back in the chair as possible (with the eventual goal of seating the patient at 90-degree hip flexion), insert the vertical slat down into the TLSO's back pocket.
2. Adjust the height of the vertical slat to position the hyperextension roll at the back of the patient's head where it seems to be most effective in assisting the patient to achieve upright posture. The vertical slat position can be readjusted later if needed.
3. The patient should feel the head/neck stretched but not uncomfortable. Do not overstretch.
4. Wait about 15-20 minutes before checking for signs of relaxation of the upper body and head/neck (in some cases it may take more time). If the patient complains or shows signs of being uncomfortable, remove the Hyperextension Stop and consult your health professional. The positioning of the Hyperextension Stop should become easier as the patient becomes used to wearing it.



Product Size	Product Code
One size	20728H

\* The length of the vertical slat is determined by the size of the TLSO ordered for both adult and pediatric patients.

## Instructions for Use

### Restorative Hyperextension Stop Cervical Attachment to TLSO

#### Cleaning Instructions:

Use hospital-approved disinfectant or household antimicrobial cleaner/disinfectant according to the label instructions for hard plastic. Padding can be laundered by hand or on gentle cycle of home type washer with mild soap. Rinse. Squeeze between towels to remove excess water. Line dry. Do not use chlorinated products such as bleach.

#### Warranty:

See [restorativemedical.com](http://restorativemedical.com) for warranty information

#### Warnings:



- This device is for single patient use only.
- Consult a clinical professional before using this device.
- Check all skin areas each time the orthosis is removed. If any discoloration does not return to normal within 3-5 minutes, discontinue use and report to your health professional. Discontinue use of the orthosis until this situation has been resolved.

#### Recommended Wear Time:

Apply the orthosis initially for 30 minutes. Remove and check the skin for redness or breakdown. If skin integrity is compromised, discontinue use and contact your clinician. Otherwise, gradually increase wearing times to the duration recommended by your clinician to no more than 6 hours at a time. Restorative Medical's wear times are suggested and should be discussed with your clinician.

#### Cautions:

- Not user repairable. Consult your clinician or Restorative Medical if damaged.
- No hazardous material present. Follow local environmental codes for waste disposal.



This product is not made with natural rubber latex (NRL) and di-(2-ethylhexyl)phthalate (DEHP).