


## Instructions for Use

### Prosperity Hand

**Indications for use:** Provides moderate, comfortable support of the hand and wrist, post injury or surgery.

#### Product Features

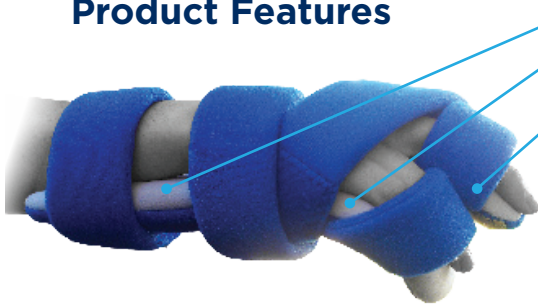

- Padding provides comfort and redistributes skin pressure.
- High-temperature plastic base can be remolded by your health professional to continue restorative process as the patient improves.
- Slit between the middle and ring fingers fits many lost-range conditions with little or no remolding.
- Thermal adaptive technology for optimal comfort.
- Lightweight support for painful extremities.
- Plastic base extends under the thumb gradually abducting it comfortably.
- Flex Technology splint moves with the patient's abnormal muscle tone and spasticity for comfort and helps to relax the abnormal muscle tone.
- Color coding aids proper strap attachment.

#### Instructions for Use:

Please read all of the instructions before use. Use as directed by your clinician. Always follow your facility protocol for skin care.

1. Locate the hand to the splint and assure comfortable thumb placement - do not overstretch.
2. Wrap the lower thumb strap over the thumb joint closest to the palm. The hand should conform to the splint with only mild resistance.
3. Feed the distal 2" strap through the space between the thumb and index fingers, over the back of the hand, and hook it to the blue loop on the back of the splint.
4. Attach the wrist strap directly over the wrist. Then attach the forearm strap to secure the arm to the splint.



5. Use the two 1" finger straps to bring the middle and index fingers toward the thumb, and the little and ring fingers in the opposite direction.



6. Attach the upper thumb strap over the thumb joint furthest from the palm or over the end of the thumb if it hyperextends.

7. Ensure the fingers and joints are properly placed and not overstretched, and there are no material folds that could make the patient uncomfortable. The patient should feel a mild stretch but no discomfort in any area.



## Instructions for Use

### Prosperity Hand IFU

Product Size	MCP Width	Wrist to Finger Length	Product Code with Left/Right
Ex-Small	2¾" to 3"	Up to 5"	20732XSL/R
Small	Up to 3"	Up to 6½"	20732SL/R
Medium	Up to 3½"	Up to 7"	20732ML/R
Large	Up to 3¾"	Up to 8"	20732LL/R

### Cleaning Instructions:

Use hospital-approved disinfectant or household antimicrobial cleaner/disinfectant according to the label instructions for hard plastic. Padding can be laundered by hand or on gentle cycle of home type washer with mild soap. Rinse. Squeeze between towels to remove excess water. Line dry. Do not use chlorinated products such as bleach.

### Warranty:

See [restorativemedical.com](http://restorativemedical.com) for warranty information.

### Warnings:



- This device is for single patient use only.
- Consult a clinical professional before using this device.
- Check all skin areas each time the splint is removed. If any discoloration does not return to normal within 3-5 minutes, discontinue use and report to your health professional. Do not use the splint until this situation has been resolved.

### Recommended Wear Times:

Apply the splint initially for 30 minutes. Remove and check the skin for redness or breakdown. If skin integrity is compromised, discontinue use and contact your clinician. Otherwise, gradually increase wearing times to the duration recommended by your clinician to no more than 6 hours at a time. Restorative Medical's wear times are suggested and should be discussed with your clinician.

### Cautions:

- Not user repairable. Consult your clinician or Restorative Medical if damaged.
- No hazardous material present. Follow local environmental codes for waste disposal.



This product is not made with natural rubber latex (NRL) and di-(2-ethylhexyl)phthalate (DEHP).



Prevent • Protect • Restore

332 East Broadway • Brandenburg, KY 40108  
(800) 793-5544 • 270-422-5454 • Fax: (270) 422-5453

[www.restorativemedical.com](http://www.restorativemedical.com)



RM- Prosperity Hand IFU v1.0 - 02/15/2017