

Instructions for Use

Restorative Shoulder

Indications for Use:

Provides stability and support for chronic shoulder pain.

Product Features:



RANGE OF MOTION HINGE:
The front and rear hinges are preset at 15°. They can be adjusted to 45° or 90°. It is recommended to adjust hinge when the brace is off.




OPTIONAL COMFORT STRAP:
If you encounter undue pressure across your neck from the Shoulder Strap, attach the Comfort Strap to help to even out weight distribution.



TELESCOPING ARM FRAME:
The brace can be adjusted to any arm length. Loosen the two screws (as shown) in order to adjust the length of the Arm Frame.

Instructions for Use:

Please read all of the instructions before use. Use as directed by your clinician. Always follow your facility protocol for skin care.

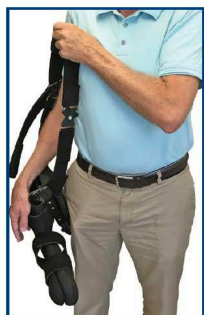

STEP 1:
While standing, place the PAKshoulder under the desired arm and hold by the Shoulder Strap (blue hook & loop).



STEP 2:
Slip the Shoulder Strap over the head, across the neck and chest.



STEP 3:
Undo the waist strap from the front of the brace. Bring around from the back. Feed the strap through the front D-ring and tighten.



STEP 4:
Slide forearm through the three straps on the Arm Frame. Place the top of the brace (two hinges) up firmly under the arm pit. Tighten arm straps.



STEP 5:
If there is too much pressure against the neck, connect the (optional) Comfort Strap (beige hook & loop) through the D-ring. Adjust the Shoulder and Comfort Straps to enhance comfort.



STEP 6:
Adjust and lock the front and back hinges to the desired position. Hinges can be readjusted as needed. (See backside for more detail to adjust hinges.)

Instructions for Use

Restorative Arm/Shoulder



The brace can be set to the desired position by adjusting the two hinges. In order to set the Range of Motion hinges, the plastic cover must be flipped open as seen in pictures to the left. The brace is pre-set at 15°. It can be adjusted to 45° or 90°. The picture on the left shows how the hinge should look for a 45° setting. The middle picture shows how the hinge should look for a 90° setting. The rightmost picture shows the different settings.

Cleaning Instructions:

ONLY hand wash the brace. Hand wash the liner and straps with mild detergent and let air dry.

Warranty:

See restorativemedical.com for warranty information.



Warnings:

- This device is for single patient use only.
- Consult a clinical professional before using this brace.
- Check all skin areas each time the brace is removed. If any discoloration does not return to normal within 3-5 minutes, discontinue use and report to your health professional. Discontinue use of the splint until this situation has been resolved.

Recommended Wear Time:

Apply the brace initially for 30 minutes. Remove and check skin for redness or breakdown. If skin integrity is compromised, discontinue use and contact your health professional. Otherwise, gradually increase wearing times to the duration recommended by your clinician to no more than 6 hours at a time. Restorative Medical's wear times are suggested and should be discussed with your health professional.

Cautions:

- Not user repairable. Consult your clinician or Restorative Medical if damaged.
- No hazardous material present. Follow local environmental codes for waste disposal.



This product is not made with natural rubber latex (NRL) and di-(2-ethylhexyl) phthalate (DEHP).



Prevent • Protect • Restore

332 East Broadway • Brandenburg, KY 40108
(800) 793-5544 • 270-422-5454 • Fax: (270) 422-5453
www.restorativemedical.com



Restorative shoulderIFU v1.0 - 12/13/2018