

Instructions for Use

Restorative Hinged Knee

Indications for Use:

Provides stability and support for chronic knee pains and instability.

Instructions for Use:

Please read all of the instructions before use. Use as directed by clinician. Always follow your facility protocol for skin care.



1. Open the knee brace and lay it flat with the label at the top of the splint. Separate the two pieces of the knee splint. (photo A) Adjust it with the hook closures to exactly fit your leg (photo B).

2. Wrap the knee brace around the knee starting from the knee's back side.



3. Line up the hinges with the center of the knee cap on both sides of the leg and secure the top and bottom closure tabs.



4. Feed the top closure strap through the top D-ring and tighten to ensure a snug fit. Do the same with the bottom-closure strap. Tighten and re-adjust as needed.

Hinge Adjustment

If ordered by your physician, the hinge can be adjusted by a clinical professional to limit the range of motion of the knee. Otherwise, do not open the cover. The controls are set to allow free movement and should not be changed.

Instructions for Use

Restorative Hinged Knee

Cleaning Instructions:

Remove hinges from brace and wipe with antimicrobial cleaner according to the label instructions. Hand wash fabric in cold water with mild detergent. Line dry. Reassemble. Do not iron, dry clean or use chlorinated products such as bleach.

Warranty:

See restorativemedical.com for warranty information.



Warnings:

- This device is for single patient use only.
- Consult a clinical professional before using this splint.
- Check all skin areas each time the splint is removed. If any discoloration does not return to normal within 3-5 minutes, discontinue use and report to your health professional. Discontinue use of the splint until this situation has been resolved.

Recommended Wear Time:

Apply the splint initially for 30 minutes. Remove and check skin for redness or breakdown. If skin integrity is compromised, discontinue use and contact your health professional. Otherwise, gradually increase wearing times to the duration recommended by your clinician to no more than 6 hours at a time. Restorative Medical's wear times are suggested and should be discussed with your health professional.

Cautions:

- Not user repairable. Consult your clinician or Restorative Medical if damaged.
- No hazardous material present. Follow local environmental codes for waste disposal.



This product is not made with natural rubber latex (NRL) and di-(2-ethylhexyl) phthalate (DEHP).



Prevent • Protect • Restore

332 East Broadway • Brandenburg, KY 40108
(800) 793-5544 • 270-422-5454 • Fax: (270) 422-5453
www.restorativemedical.com



Restorative Hinged KneeIFU v1.01-10/22/2019