

Instructions for Use


Restorative OA Knee

Indications for Use:

Provides stability and support for the knee.

Instructions for Use:

Please read all of the instructions before use. Use as directed by your clinician. Always follow your facility protocol for skin care.

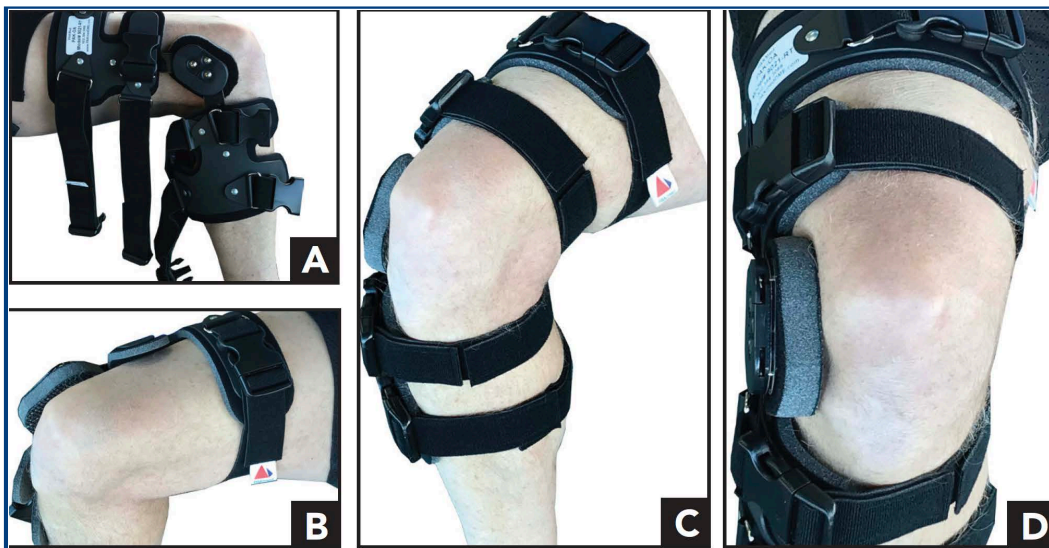

STEP 1: Loosen the straps from the buckles and place the knee brace so that the hinge aligns with the center of the knee (Fig A).

STEP 2: Beginning at the top buckle, loosely thread each strap through the front D-ring, keeping the hinge aligned with the knee (Fig B).

STEP 3: Continue working down the knee brace, loosely threading all four of the straps (Fig C).

STEP 4: With the knee bent, tighten each strap so that the brace is snug and does not slide or shift (Fig D).

Instructions for Use

Restorative OA Knee



HINGE ADJUSTMENT

If your doctor or medical professional recommends limiting the range of motion of your knee, this hinge can be adjusted to accommodate any restrictions.

- 1) Open the cover to the hinge.
- 2) Adjust the pins to the desired degree of flexion or extension.
- 3) Close the cover to the hinge.

Cleaning Instructions:

ONLY hand wash the brace. Hand wash the liner and straps with mild detergent and let air dry.

Warranty:

See restorativemedical.com for warranty information.



Warnings:

- This device is for single patient use only.
- Consult a clinical professional before using this brace.
- Check all skin areas each time the brace is removed. If any discoloration does not return to normal within 3-5 minutes, discontinue use and report to your health professional. Discontinue use of the brace until this situation has been resolved.

Recommended Wear Time:

Apply the brace initially for 30 minutes. Remove and check skin for redness or breakdown. If skin integrity is compromised, discontinue use and contact your health professional. Otherwise, gradually increase wearing times to the duration recommended by your clinician to no more than 6 hours at a time. Restorative Medical's wear times are suggested and should be discussed with your health professional.

Cautions:

- Not user repairable. Consult your clinician or Restorative Medical if damaged.
- No hazardous material present. Follow local environmental codes for waste disposal.



This product is not made with natural rubber latex (NRL) and di-(2-ethylhexyl) phthalate (DEHP).



Prevent • Protect • Restore

332 East Broadway • Brandenburg, KY 40108
 (800) 793-5544 • 270-422-5454 • Fax: (270) 422-5453
www.restorativemedical.com



Restorative OA Knee v1.0 - 12/21/2018