

Instructions for Use

Restorative Wrist

Indications for Use:

Provides stability and support for the wrist.

Product Features:

**AMBIDEXTROUS DESIGN:**

The wrist brace can be worn on either the right or left wrist/hand.

OPTIONAL FINGER SEPARATOR: The Finger Separator can be attached to the cone if pressure points occur due to fingers rubbing against each other.

PASSIVE RANGE OF MOTION (PROM) HINGE:

The PROM hinge can be adjusted to the degree of flexion/extension desired (see picture insert). Use a Phillips screwdriver to achieve the chosen degree.

Instructions for Use:

Please read all of the instructions before use. Use as directed by your clinician. Always follow your facility protocol for skin care.



STEP 1: Adjust the Passive Range of Motion (PROM) hinge to allow the maximum wrist and finger extension.

STEP 2: Place fingers around cone at end of orthosis and forearm on the concave surface.

STEP 3: Tighten the three straps so they are snug. Do not over tighten.

STEP 4: Attach the Finger Separator to the cone if desired using hook & loop.

STEP 5: Check the degree of the flexion/extension to ensure it is adjusted properly. It can be adjusted and readjusted as needed.

Instructions for Use

Restorative Wrist

Cleaning Instructions:

ONLY hand wash the brace. Hand wash the liner and straps with mild detergent and let air dry.

Warranty:

See restorativemedical.com for warranty information.



Warnings:

- This device is for single patient use only.
- Consult a clinical professional before using this brace.
- Check all skin areas each time the brace is removed. If any discoloration does not return to normal within 3-5 minutes, discontinue use and report to your health professional. Discontinue use of the brace until this situation has been resolved.

Recommended Wear Time:

Apply the brace initially for 30 minutes. Remove and check skin for redness or breakdown. If skin integrity is compromised, discontinue use and contact your health professional. Otherwise, gradually increase wearing times to the duration recommended by your clinician to no more than 6 hours at a time. Restorative Medical's wear times are suggested and should be discussed with your health professional.

Cautions:

- Not user repairable. Consult your clinician or Restorative Medical if damaged.
- No hazardous material present. Follow local environmental codes for waste disposal.



This product is not made with natural rubber latex (NRL) and di-(2-ethylhexyl) phthalate (DEHP).



Prevent • Protect • Restore

332 East Broadway • Brandenburg, KY 40108
(800) 793-5544 • 270-422-5454 • Fax: (270) 422-5453
www.restorativemedical.com



Restorative WristIFU v1.0 - 12/13/2018